

schülke -+



Alcohol-free wipes for the cleaning and disinfection of alcohol-sensitive surfaces and medical devices.

Alcohol – Free

mikrozid® sensitive wipes

Our Plus

- alcohol-free
- ready-to-use
- very fast-acting and effective
- for alcohol-sensitive surfaces
- good cleaning performance

Product profile

Mikrozid® sensitive wipes is an alcohol-free disinfectant wipes that comes with a dual action cleaning effect. Due to its excellent material compatibility, it can be used on alcohol-sensitive surfaces and medical devices.

Application areas

Medical equipment as defined by the Medical Device Directive (MDD), such as:

- surfaces of medical devices, such as a stethoscope.
- application parts, such as ultrasonic heads of probes for abdominal examinations or acrylic glass.
- alcohol-sensitive application parts that cannot be submerged in liquids, such as ultrasonic heads or probes for abdominal examinations. Please take note that for the disinfection of semi-critical devices (e.g. ultrasonic heads of probes for vaginal examinations), a high-level disinfectant is required.

For surfaces according to the Biocidal Products Directive, such as:

- small surfaces in hospitals, medical and dental practices and ambulances.
- small surfaces in medical laboratories, residential and nursing homes.

Instructions for use

Wipe the surface thoroughly and ensure that a complete wetting is achieved. For norovirus, please keep the surface moist during the entire exposure period. Make sure that all visible dirt is removed prior to disinfection. When opening and re-inserting the first cloth to the latch, make sure that good hygiene requirements are maintained. Remember to close the lid tightly after use. With the correct way of use, Schülke guarantees a shelf-life of 3 months after the package is opened.

Microbiological efficacy

Efficacy	Concentration	Contact time
bactericidal <i>in accordance with VAH Guidelines with short contact times</i>	ready-to-use	1 min.
MRSA, ESBL	ready-to-use	1 min.
levurocidal <i>in accordance with VAH Guidelines with short contact times</i>	ready-to-use	1 min.
Mycobacteria (Including TB) EN14348 <i>- dirty conditions</i>	ready-to-use	60 min.
enveloped viruses (incl. HIV, HBV, HCV) <i>in accordance with DVV (German Registered Association for Combating Viral Diseases)//RKI Guideline</i>	ready-to-use	1 min.
Norovirus EN14476	ready-to-use	15 min.
Polyoma SV40	ready-to-use	1 min.
Rotavirus	ready-to-use	1 min.

Certificates

- IHO listed
- VAH certificate



CE 0297



mikrozyd® sensitive wipes

Product Data

Composition:

100 g agent solution contain the following active ingredients: 0.26 g Alkyl(C12-C16) dimethylbenzyl ammonium chloride (ADBAC/BKC (C12-C16)); 0.26 g Didecyldimethylammonium chloride (DDAC), 0.26 g Alkyl(C12-C14) ethylbenzylammonium chloride (ADEBAC (C12-C14))

Chemical-physical data	
Color	colourless
Density	ca. 1g/cm ³ / 20 °C
Flash point	not applicable
Form	liquid
pH	6 - 8 / 20 °C / concentrate
Viscosity, dynamic	not determined

Special advice

Use disinfectants safely. Always read the label and product information before use.

The suitability of the product towards the efficiency spectrum must be tested individually. The following applies for the choice of surface disinfectants: "Surface disinfectants distinguish themselves among other things by their effectiveness towards microbes. This must be remembered during the choice of disinfectants for individual areas." Please remember that some bacterial strains are less vulnerable or resistant towards disinfectants than others. **It is also possible that products based on quaternary ammonium compounds are insufficient. Due to this, Schülke recommends to use mikrozyd® PAA wipes or mikrozyd® AF wipes for disinfection of surfaces in high-risk areas (for example: intensive care, haemato-oncology or neonatology) as these products have a higher effectiveness. Not suitable for the disinfection of invasive medical devices.

Available Size

Item	Size	*Surface coverage
Mikrozyd® sensitive wipes Premium	6 x 100 wipes (20 x 20 cm)	Approx. 1.0 m ²

Information for order

Item	Delivery form	Item no.
mikrozyd® sensitive wipes premium (100 wipes)	6 / carton	70000807

Environmental information

Schülke manufactures products economically and with advanced, safe and environmentally friendly production processes while at the same time maintaining out high quality standards.

Expert opinion and information

Please visit our website for an overview of all available literature/ reports on the product:

Singapore: www.schuelke.com/sg-en/

Malaysia: www.schuelke.com/my-en/

*The surface coverage of one wipe depends on the surrounding conditions and the structure of the wiped surface.

**Recommendation from RKI (The public health institute for Germany): Anforderungen an die Hygiene bei der Reinigung und Desinfektion von Flächen.



Schülke & Mayr GmbH holds a Manufacturer's Authorisation according to sect 13 para 1 German Drug Law and Certificates of GMP Compliance for medicinal products.



A company of the Air Liquide Group

Schülke Headquarters
Schülke & Mayr GmbH
Robert-Koch-Str. 2
22851 Norderstedt
Germany
Phone +49 40 52100-0
Fax +49 40 52100-318
www.schuelke.com
info@schuelke.com

Schülke & Mayr (Asia) Pte Ltd
Sime Darby Enterprise Centre
10 Jalan Kilang #04-01/02/03
Singapore 159410
Phone +65 6257 2388
Fax +65 6257 9388
mail.sg@schuelke.com

Schülke & Mayr (Asia) Sdn Bhd
Block A, Plaza Glomac
Lot A-06-01 & A-06-02
No 6 Jalan SS7/19, Kelana Jaya
47301 Petaling Jaya
Selangor, Malaysia
Phone +603 7885 8020
Fax +603 7885 8021
mail.my@schuelke.com

Imported and distributed by:

Medisafe Philippines, Incorporated
Suites 303 & 310 West City Plaza Building 66
West Avenue, Quezon City PH 1104
Phone +63 8374 8910 | Telefax +63 3416 1874
Email: customercare@medisafe.com.ph
www.medisafe.com.ph

& | medisafePH